



An 044 Wire is inserted through the base of the Tunnel Pivots and welded to the frame, unlike surface weld or solder designs which can come apart with force. Apple Core-Type Hex Screws are used to secure the hinges and Ceka Bond is used to minimize loosening.

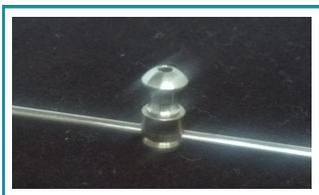

The **Dream Systems Herbst Oral Appliance™** is FDA and Medicare Cleared for Sleep Apnea. It incorporates a Telescopic Hinge with Advancement to 5mm. One turn of the advancement mechanism (until the next hole appears) advances the mandible about 1/16 mm. The arrows indicate the directions of advancement; Right side-up, Left side-down.

There are **Medicare Two Hinge Options: Single Nut Hinge- and the Double Nut Stabilizing Hinge.** Ball Clasps, plus Hooks and Elastics are standard.

There are **Two Framework Designs: Medicare Welded Buccal Framework** and our **Premium Design with a Fully Welded Buccal and Lingual Framework.**

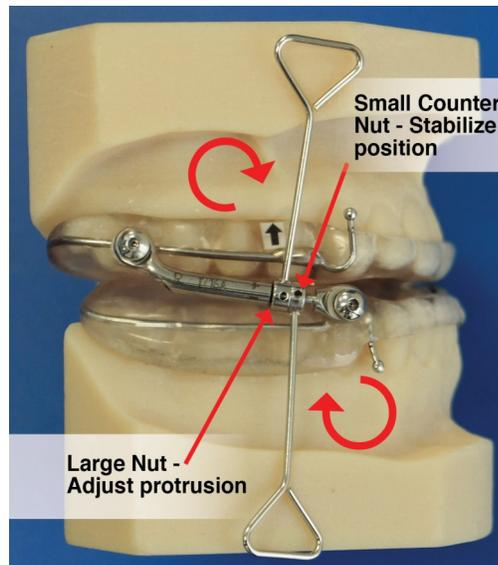

TELESCOPIC  
 DOUBLE NUT  
 SCREW DESIGN  
 WITH  
 ADJUSTMENT KEY