



The **O2 OASYS Oral / Nasal Airway System** is cleared as a 510 K Medical Device (#K030440) by the ENT and Dental Divisions of the FDA: 1) Mandibular Repositioner for treatment of Snoring and Sleep Apnea; 2) Nasal Dilator for improved breathing through the nose. It also treats Sleep Disordered Breathing by addressing Upper Airway Resistance in the nasal region. The Lingual Tongue Button component is optional, but adds tongue training and repositioning for improved tongue placement, better swallowing and improved sleep treatment results,**even in Severe OSA patients.**

Three Functions in One Device

- ◆ Mandibular Advancement
- ◆ Improved Nasal Breathing
- ◆ Improved Tongue Position

The Anterior Labial Shield adjusts easily forward and backward, **8-15mm** on parallel bars, with an easy locking system and a **MM Scale**. A Clear Splint acts as a cushion to reduce pressure on the upper anterior teeth and being unattached, provides greater freedom of movement. The Upper Oasys splint can be a full coverage or open lingually from #7-10, to encourage tongue placement at the incisal papilla. **Patented Nasal Dilators** increase airflow through the nose with internal extensions in the maxillary vestibule that stretch the nasiolabial fold. Optional **Patented Lingual Tongue Buttons** extend off the lower splint in the posterior molar region to guide the tongue up and forward to the palate. This also makes the patient's swallowing more dynamic, establishing a better tongue posture and improving results in even patients with Severe OSA

Choose the OASYS for...

- Patients with Nasal Obstruction or Resistance
- Patient Wearing an Upper Denture
- Patient Missing Upper Posterior Teeth with only Dentition Cuspid to Cuspid
- Patient Missing Lower Posterior Molars
- Patients also in Invisalign Treatment
- Moderate-Severe Cases where Mandibular Advancement alone could limit effectiveness.

John Bixby, DMD—Sleep Dynamics—NJ

M/F	AHI BEFORE	PSG	AHI AFTER
F	77.1		5.6
M	120.3		10.0
F	48.8		2.4
F	88.2		8.3
M	96.3		1.9
F	65.0		4.5
M	67.7		4.2
M	82.0		0.0
F	67.6		3.2
M	45.0		4.5

(*164 Patients with the OASYS with Nasal Dilators & Tongue Repositioning Buttons)




OASYS Simplicity



Simplicity in Mandibular Advancement - Sliding Shield with Locks and Measurement Scales. No Hinges, Fins or Hooks!

The Cottle Maneuver

Evaluate how Nasal Dilation can enhance Mandibular Advancement results by opening the nasal passages.

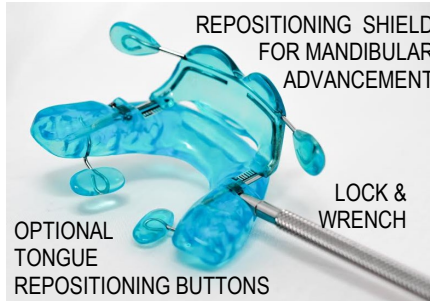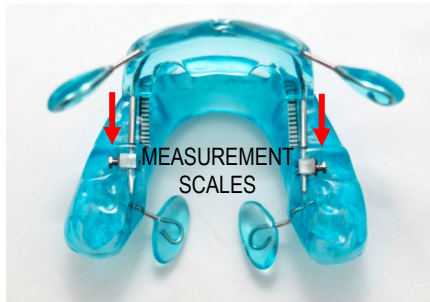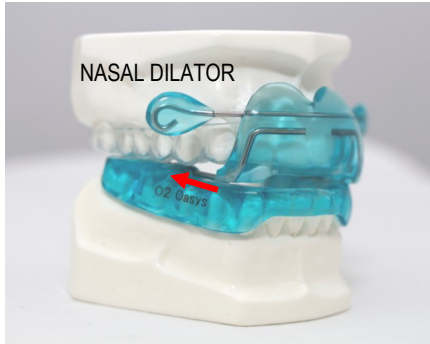

720 Sunrise Avenue, #230-A
Roseville, CA 95661
(916) 865-4528

www.dreamsystemsdentallab.com