





The **OASYS HERBST Appliance** is **FDA (#K083209)** and **Medicare Compliant (#M001)**, for Sleep Apnea. The side OASYS hinges provide 10mm of Advancement and the Screws can be opened 1-2mm for Retraction. They hold the lower jaw forward during sleep, but allows the patient to open and close his/her mouth and have some side to side movement. This will help to open the airway in back of the throat, while pulling the tongue forward. There are two framework designs: **Medicare Partially-Welded Framework** and a **Premium-Welded Framework design are available**. Ball Clasps, plus Hooks and Elastics are standard. **Medicare** has also **Cleared** the addition of **Nasal Dilators (#M002)** and with **Lingual Tongue Repositioning Lifters (#M003)** on the OASYS Herbst Appliance.




10 MM Advancement

Easy Advancement & Monitoring
Set Open 1-2mm for Retraction
Shorter Length for Better Placement

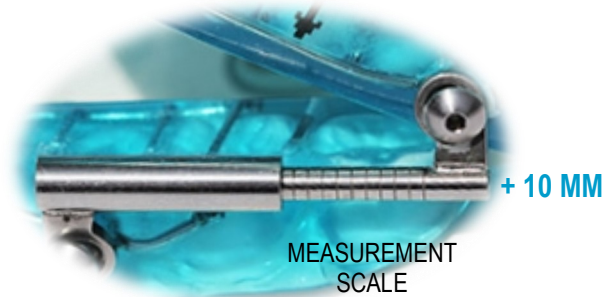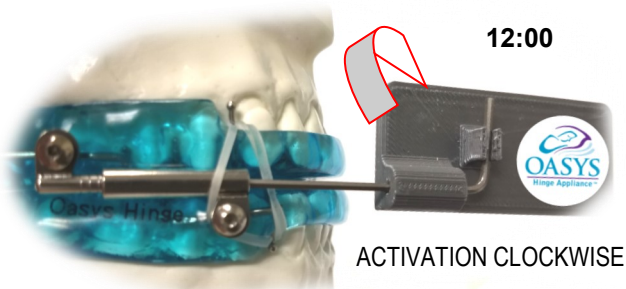


12:00 TO 6:00 = 0.25 MM 1/2 TURN
12:00 TO 12:00 = 0.50 MM FULL TURN
2 REVOLUTIONS = 1.00 MM TWO FULL TURNS

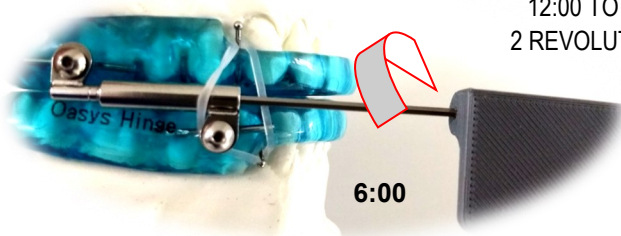

INTRA ORAL ACTIVATION



#M002 IMPROVE NASAL BREATHING WITH NASAL DILATORS

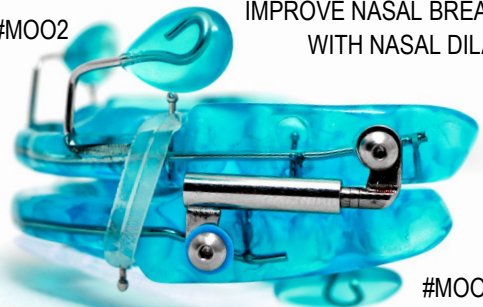

IMPROVE RESULTS IN MODERATE-SEVERE OSA PATIENTS BY TRAINING THE TONGUE WITH TONGUE REPOSITIONING LIFTERS